



## ROGYPAL AC-308

Rinse cleaning compound

- **ROGYPAL AC-308** is specially developed rinse-cleaning compound for it, after cleansing with one of our alkaline cleaning concentrates, rinse and neutralize instruments in coil washing machines.
- Removes scale, oxides, alkaline traces, etc.
- Biodegradable.

### Where to use

Clear, liquid, odourless and colourless, acid rinse-cleaning compound for cleaning, rinsing and neutralizing of instruments with the aid of a washing machine at the CSSD in hospitals.

Safe for steel, stainless steel, zinc, glass, silver, porcelain and plastics..

### Packing

Box of 2 x 5,5 kg

### How to use

Spray Washers:

0.5% **ROGYPAL AC-308** by volume. Temp .: 20-60 ° C.

overall:

1- 2% by volume in water. Temp .: 20-80 ° C

### Properties

physical aspect	: clear liquid
pH at working concentration	: 1.5 ± 0.5
Odor	: odourless
Solubility in water	: full

### Safety

The safety information of this product is mentioned in our Safety Data Sheet (MSDS).  
If using cleaning- and/or disinfectant products use always personal protective equipment

\* Only to be used by professional users.