



LUBRI-KLEEN

COMPOUND FOR BRIGHTENING AND LUBRIFYING

- Instruments dry after rinsing without any stains or stripes, so wiping down with a cloth is not necessary.
- **LUBRI-KLEEN** is especially designed to be used as a rinse in instrument washing machines on any desired temperature .
- Because of special the deep penetrating wetting agents **LUBRI-KLEEN** lubricates all hinges of the instruments, which guarantees they'll stay flexible.
- **Biodegradable.**
- **Phosphate-free.**

Where to use

Clear, amber, liquid brightener and lubricant for mechanically rinsing all sorts of instruments on central sterilization departments, operating rooms and hospital laboratories.
Harmless to steel, stainless steel, porcelain and plastics

Packing

Carton of 2 x 5 ltr

How to use

Application and dosage:

When using a spray washing machine the dosage of **LUBRI-KLEEN** is programmed and dosed automatically at the end of each washing program. Per rinsing 0,04 - 0,2% **LUBRI-KLEEN** by volume of water is dosed.

Properties

Physical aspect	:	amber liquid
pH at working concentration	:	8,0 ± 0,5
Odor	:	fresh
Solubility in water	:	complete

The safety information of this product is mentioned in our Safety Data Sheet (MSDS).
If using cleaning- and/or disinfectant products use always personal protective equipment.

* Only to be used by professional users.